Aquatec® Sorrento

- EN Shower stool / Shower chair**
User Manual
- DE Duschhocker / Duschstuhl**
Gebrauchsanweisung
- FR Tabouret de douche / Chaise de douche**
Mode d'emploi
- IT Sgabello per la doccia / Sedia da doccia**
Istruzioni per l'uso
- ES Taburete de ducha / Silla de ducha**
Manual de instrucciones
- NL Douchekrak / Douchestoel**
Gebruiksaanwijzing
- DA Badetaburet / Brusestol**
Brugsanvisning
- FI Suihkujakkara**
Käyttöohje
- NO Dusjkrakk / Dusjstol**
Brukerveiledning
- SV Duschpall / Duschstol**
Bruksanvisning
- PT Banco de duche / Cadeira para duche**
Manual de instruções

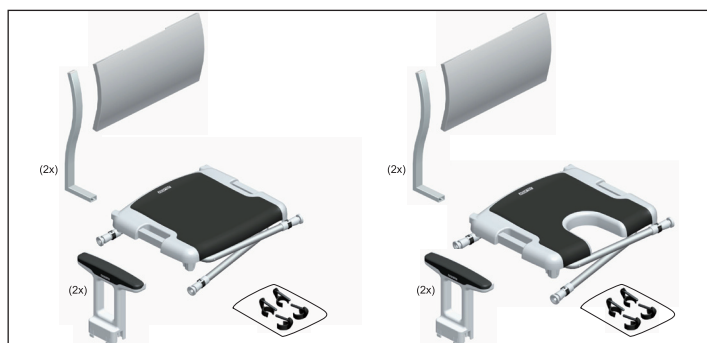
Fig. 1

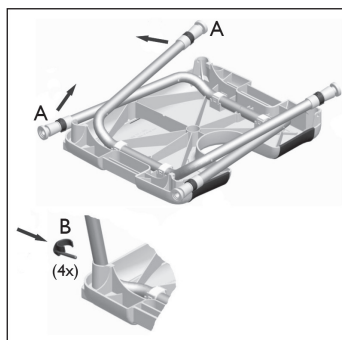
Fig. 2

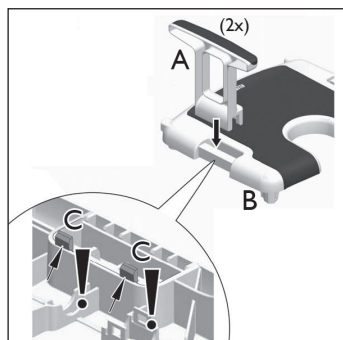
Fig. 3

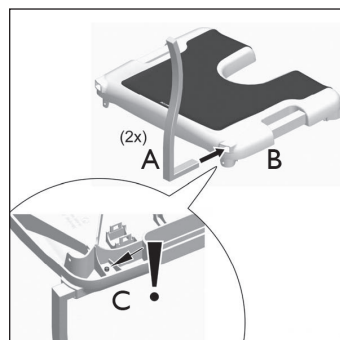
Fig. 4

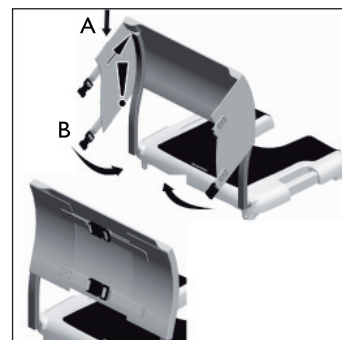
Fig. 5

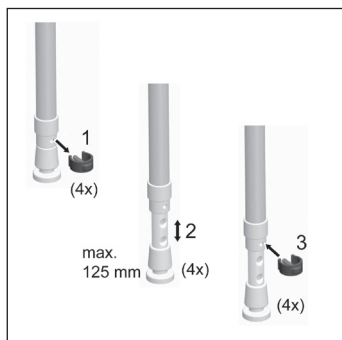
Fig. 6

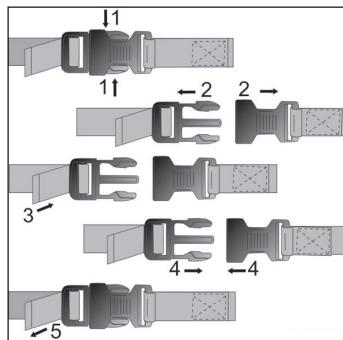
Fig. 7

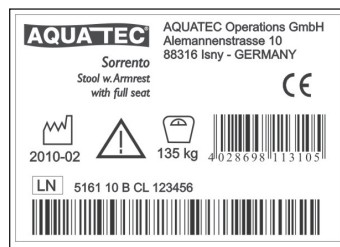
Fig. 8

This manual must be given to the user of the product.
Before using the product, read this manual and save it
for future reference!

AQUATEC®

Yes, you can.

Intended Use

This product is designed solely as an aid for taking a shower and for use in the bathroom. Any other use is prohibited.

WARNING Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.

Operating temperature: 10 - 40 °C

Max. user weight: 135 kg

Service Life

The expected service life of this product is three years when used in accordance with safety instructions, maintenance intervals and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Safety

WARNING Risk of injury!

- Do not use if defective.

Scope of delivery

- Fig. 1.

Assembly

CAUTION Risk of entrapment!

- Be careful during assembly.

- Fig. 2 - 5.

Use

- Fig. 6, 7.

Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined above are adhered to.

- Check product regularly for damage and to ensure that it is securely assembled.

Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

- Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG002 and your local infection control policy.

- Clean by hand.
- Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
- Max. cleaning temperature 85 °C for 3 minutes.

Re-Use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean the product thoroughly and check the safety functions, reset if necessary.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulations.

Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business. Guarantee claims can only be made through the relevant specialist dealer.

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Invacare AB, Fagerstagatan 9,
Aquatec Operations GmbH