beurer

MG 300 HD-XL

- | | |
|---|---|
| (D) Shiatsu-Sitzauflage
Gebrauchsanweisung..... 2 | (TR) Koltuk üstü shiatsu
Kullanım kılavuzu..... 38 |
| (GB) Shiatsu seat cover
Instructions for use 10 | (RUS) Массажная накладка шиацу на
сиденье
Инструкция по
применению 45 |
| (F) Siège de massage Shiatsu
Mode d'emploi 17 | (PL) Shiatsu – masażer na siedzenie
Instrukcja obsługi..... 53 |
| (E) Colchoneta de asiento Shiatsu
Manual de instrucciones..... 24 | |
| (I) Coprisedile shiatsu
Istruzioni per l'uso 31 | |

Inhalt

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Germany

Contents

1. About Shiatsu massage	11	6. Start-up	14
2. Getting to know your appliance	11	7. Operation.....	14
3. Proper use	11	8. Care and storage	15
4. Safety information	11	9. In the event of problems.....	16
5. Appliance description.....	13		

Items included in the package

- Shiatsu seat cover
- These operating instructions

WARNING

- The unit is only intended for domestic/private use, not for commercial use.
- This device may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely, and are fully aware of the consequent risks of use.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.

Explanation of symbols

The following symbols are used in the operating instructions and on the type plate:

WARNING Warning of risks of injury or health hazards.

CAUTION Safety information about possible damage to appliance/accessories.

Note Important information.

The device has double protective insulation and therefore complies with protection class 2.

Only use indoors.

1. About Shiatsu massage

Shiatsu is a form of body massage which was originally developed in Japan around 100 years ago. It is based on ideas anchored in traditional Chinese medicine, in particular the meridian system, i.e. the paths in the human body along which energy is conveyed.

The aim of any Shiatsu massage is to promote the physical, emotional and mental wellbeing of the person being treated. To do this, energy blocks and bottlenecks in the meridians need to be dissolved and the self-regulating ability of the body needs to be stimulated.

Shiatsu therapists achieve this by exercising pressure in flowing movements along the meridians. Although Shiatsu can be literally translated as "finger pressure", practitioners also use the heels of their hands, their elbows and knees as well as their fingers. In the process, the therapist will use his or her full body weight in order to generate the required pressure.

2. Getting to know your appliance

The massage heads in your Shiatsu massager imitate the pressure and kneading movements of a Shiatsu massage. This device with its moving massage heads provides a powerful and intense deep massage for the whole back and neck. You have to place the Shiatsu seat cover on a suitable seat pad for this purpose. The device is only for **private use** and is not intended for medical or commercial purposes.

3. Proper use

This device is solely intended for massaging the backs of human beings. It cannot replace medical treatment. Do not use the massage device if any of the following warnings apply to you.

WARNING

Use of the massage device is prohibited in the following cases:

- Not with changes caused by disease or injury in the back region (e.g. disc problems, open wounds),
- not during pregnancy,
- never while you sleep,
- not in the car,
- not on animals,
- not during activities during which an unexpected reaction may be dangerous,
- not while ingesting substances that reduce reaction times (e.g. painkiller medications, alcohol),
- never longer than 15 minutes (risk of overheating) and allow it to cool off for at least 15 minutes before using it again.

Please consult your doctor if you are unsure whether or not the massage device is suitable for you.

The device has an adjustable heating function in the massage heads. Persons who do not feel heat must be careful when using this function.

You should consult your doctor before using the massage device if:

- you suffer from a serious illness or are recovering from an operation on the upper body,
- you are fitted with a cardiac pacemaker, implant or other aid,
- Thromboses,
- Diabetes,
- in case of pains of unclarified origin.

The device is only intended for the purpose described in these operating instructions. The manufacturer cannot be held liable for damage occurring as a result of inappropriate or careless use.

4. Safety information

Please read these operating instructions carefully. Failure to observe the information provided below can lead to personal injury or material damage. Keep these instructions in a safe place where they can also be read by other persons using the device. Always pass on these instructions together with the appliance if you sell it or pass it on.

WARNING

Keep children away from the packaging materials. Risk of suffocation.

WARNING: Electric shock

Just like any other electrical appliance, this massage device must be handled with due care and attention in order to prevent the risk of electric shock.

For this reason, the device must only be operated as follows:

- With the mains voltage specified on the device only,
- Do not use the device if damage to the device itself or any of its accessories is evident.
- Never use the appliance during a thunderstorm.

In the event of a defect or malfunction, switch the device off immediately and disconnect it from the power supply. Never pull on the power cable or on the appliance in order to disconnect the plug from the mains outlet. Never hold or carry the device by the power cable. Keep the cables away from hot surfaces. Do not pinch, kink or twist the cables. Do not stick in any pins, needles or sharp objects.

Make sure that the massage device, the switch, the power supply unit and the cable do not come into contact with water, steam or other fluids.

For this reason, the device must only be operated as follows:

- Only in dry inside rooms (e.g. never in the bathtub or sauna).
- Only with dry hands.

Never try to retrieve the device if it falls into water. Disconnect the plug immediately from the mains outlet. Do not use the device if damage to the device itself or any of its accessories is evident. Do not subject the device to any impact and do not drop it.

Repairs

WARNING

- Repairs to electrical appliances must only be carried out by qualified persons. Incorrect repairs can lead to considerable danger for the user. In the event of repairs, please contact our customer service or an authorized dealer.
- The zip fastener on the Shiatsu massage device must not be opened. It is only fitted for reasons relating to the manufacturing of the device.
- If the power cord is damaged it must be replaced by an authorised dealer.

WARNING: Fire hazard

There is a risk of fire if the device is used incorrectly or if it is not used according to these operating instructions.

For this reason, the massage device must only be operated as follows:

- Never use the device under covers, such as blankets or pillows.
- Never use the device in the vicinity of petrol or other easily flammable materials.

Handling

CAUTION

There is a risk of fire if the device is used incorrectly or if it is not used according to these operating instructions.

- Do not insert any objects in the device apertures or in the moving parts. Ensure that the moving parts can move freely at all times.
- Operate the device in an upright position only, never place it flat on the floor.
- Never sit or stand on the moving parts of the device with all your weight and do not place objects on the device.
- Keep the device away from high temperatures.

Disposal

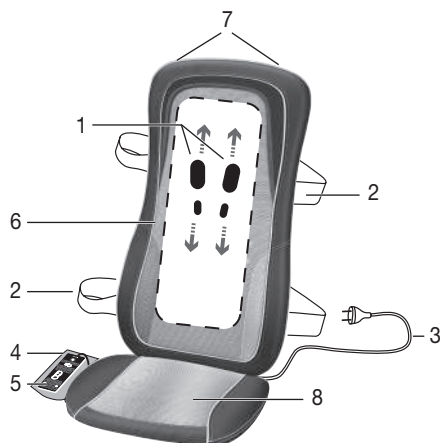
CAUTION

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.


5. Appliance description

Shiatsu massage pillow (front)

- 1 4 massage rollers
- 2 Fastening straps
- 3 Mains cable
- 4 Storage pouch
- 5 Hand switch
- 6 Removable and washable cover
- 7 Coat hanger loops (rear)
- 8 Removable seat cushion

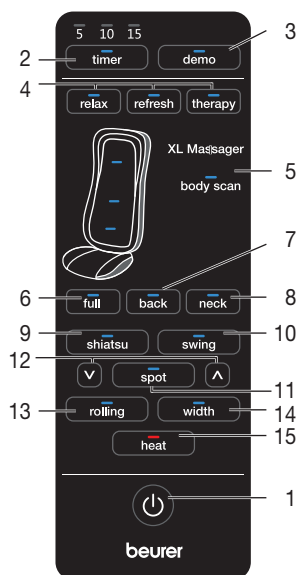Hand switch

- 1 Button
- 2 **timer** button (5–10–15 minutes)
- 3 **demo** button
- 4 Automatic massage programs
- 5 LED display for body scanning
- 6 **full** button (massage for the whole back and neck area)
- 7 **back** button (massage for the whole back area)
- 8 **neck** button (massage for the neck area)
- 9 **shiatsu** button (Shiatsu massage)
- 10 **swing** button (swing massage)
- 11 **spot** button (spot massage)
- 12 height adjustment button (spot massage)
- 13 **rolling** button (rolling massage)
- 14 **width** button (distance between massage rollers)
- 15 **heat** button (in-seat heating)

6. Start-up

CAUTION

- **IMPORTANT: Before using the device for the first time, ensure you have released all transport locks on the massage mechanism. To do this, remove the screws using the tool provided on the rear of the seat cover. If the transport lock screws are not removed first and the Shiatsu seat cover is used immediately, the device will become irreparably damaged and unusable. The warranty will not apply in this instance.**
- Reuse the transport lock screws when transporting the device in the future. Transporting the device without using the transport locks could cause irreparable damage to the device.
- Position the seat cover upright on a suitable seat with a sitting area and backrest (chair, sofa or similar). Make sure that the seat has a sufficiently high backrest.
- You can achieve the most comfortable massage position on a chair with an upright back.
- Use the flexible fastening straps and hook-and-loop fasteners to secure the Shiatsu seat cover.
- Connect the device to the mains by plugging the plug into the socket.

7. Operation

Press your back against the massager in an upright posture. The upright posture is necessary to enable accurate scanning of your back. The body scanning function is used for this and determines the size of the user. The massage must be pleasant and relaxing at all times. Stop the massage or change your position or the contact pressure if you find it is painful or unpleasant.

- Do not force or jam any part of your body, in particular your fingers, between the moving massage elements or their holders in the device.
- Sit comfortably and upright on the seat cover. Ensure you sit centered and such that the massage heads can move to the left and right of your spinal column.
- Switch on the massager with the  button. The body scanning LED display flashes while your back is being scanned. After scanning, the relax massage program is activated. The LEDs signal the massage functions which are currently selected.
- The automatic switch-off function is activated with selection of the massage region and is preset to 15 minutes. You can set the automatic switch-off function to 5, 10 or 15 minutes by pressing the **timer** button. Please note that changing the timer setting makes the timer start to count again from the beginning. To avoid overheating the device, select the maximum massage duration at the beginning of the massage and do not change it again during the massage.

You can specify the settings as follows:

1. Automatic massage programs

Press the **relax**, **refresh** or **therapy** button.

relax program:

A combination of swing and rolling massage massages your back and neck area.

refresh program:

A combination of Shiatsu and rolling massage massages your back and neck area.

therapy program:

A combination of Shiatsu, rolling and swing massage massages your back and neck area.

2. Shiatsu back and neck massage

Press the **shiatsu** button. Then use the **full**, **back** or **neck** button to select the desired massage area.

3. Swing back and neck massage

Press the **swing** button. Then use the **full**, **back** or **neck** button to select the desired massage area.

You can use the **spot** button to select a **spot massage**, i.e. Shiatsu and swing massage without vertical movement. You can change the position of the spot massage by holding down one of the arrows next to the spot button until the desired position is reached.

4. Rolling massage

Press the **rolling** button. Then use the **full**, **back** or **neck** button to select the desired massage area.

During the rolling massage you can use the **width** button to select the distance between the massage rollers.

If you change the massage type (Shiatsu, swing or rolling massage) the area that was previously selected is retained. Please note that when you change from spot Shiatsu or swing massage to rolling massage, you have to select the massage area again.

5. Heating function

Press the **heat** [15] button to switch on the heating function of the massage heads. The massage heads light up red at this time.

- Do not use the massage device for longer than 15 minutes and then allow it to cool. Using the device for longer can over-stimulate the muscles and cause tension instead of relaxation. The massage can be finished at any time by pressing the  button. Please note that on switching off, the massage heads first return to the starting position at the lumbar region. This is indicated by the LED flashing. Once this position is reached, the device switches off.
- If the device is stopped by pressing a massage area button twice, it also returns to the starting position after 15 minutes and then switches off.
- Do not disconnect the device from the mains supply during operation. Always switch it off first and wait until it has returned to the initial position.

Note

- The device has an automatic shut-off mechanism set to a maximum operating period of 15 minutes.

8. Care and storage

Cleaning

WARNING

- Disconnect the device before cleaning it.
- Only clean the appliance in the manner specified. Fluid must never be allowed to enter into the device or accessories.
- Smaller stains can be removed using a cloth or moist sponge together with a little liquid detergent for delicate fabrics if necessary. Do not use any cleaning agents containing solvents.
- The removable cover is machine-washable at 40°C. Please note the symbols for care and cleaning on the label sewn into the cover.
The back mesh cover is attached with a zip above the neck area.
- Do not use the device again until it is completely dry.

Storage

If you do not use the device for longer periods, we recommend storing it in the original packaging in a dry place without weighting it down.

Alternatively, you can hang the device in a wardrobe with a coat hanger. To do so, the coat hanger must be threaded through the hanging loops on the rear side of the device.

Ensure that you use a stable clothes hanger made from wood that is able to carry the weight of the device.

CAUTION

- If you use the transport lock screws, take care to remove these before starting the device when you next use it.

9. In the event of problems

Problem	Cause	Remedy
Massage heads are moving slowly.	The load on the massage heads is too great.	Reduce the contact pressure on the head.
Massage heads are moving in the upper or lower area only.	The massage has been activated for the limited region.	Press the full , back or neck button to change or extend the area.
Massage heads not moving.	Appliance not connected to the power supply.	Connect the plug to the mains supply and switch on the appliance.
	Device not switched on.	Switch the device on with  . Select the massage region.
	Transport locks were not removed. Massage mechanism is damaged.	Contact your authorized dealer or the service address.
	Overheating protection tripped, device is damaged.	Contact your authorized dealer or the service address.

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