beurer

FT 09/1

CE

0483

BEURER GmbH • Söflinger Str. 218 • 89077 Ulm (Germany)

www.beurer.com • www.beurer-gesundheitsratgeber.com

www.beurer-healthguide.com

DEUTSCH

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ENGLISH

Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and accessories:

	Caution
	Follow instructions for use
	Type BF applied part
	Disposal in accordance with EC Directive WEEE (Waste Electrical and Electronic Equipment)
	Manufacturer
IP 27	Protected against foreign objects equal to or greater than 12.5 mm and against temporary immersion in water.
LOTI	Batch designation
	Permissible storage and transport temperature and humidity

CE

0483

BEURER GmbH • Söflinger Str. 218 • 89077 Ulm (Germany)

www.beurer.com • www.beurer-healthguide.com

Operating

Permissible operating temperature and humidity

CE

0483

BEURER GmbH • Söflinger Str. 218 • 89077 Ulm (Germany)

www.beurer.com • www.beurer-healthguide.com

Important safety information – retain for later use.

In order to make optimum use of all the merits of the clinical thermometer, you should carefully read the operating instructions before using it, retain the instructions for later use and also keep them accessible for other users.

• The thermometer is intended to measure the temperature of the human body.

• The thermometer is only designed for the measuring area on the human body stated in the instructions for use.

• Children should not use the thermometer without supervision.

• Before each use, check the thermometer for signs of damage or wear. Damaged or worn thermometers should not be used.

• Always wait for the acoustic signal which indicates the minimum time required for measurement. Discuss the temperature with your family physician.

beurer

FT 09/1

CE

0483

BEURER GmbH • Söflinger Str. 218 • 89077 Ulm (Germany)

www.beurer.com • www.beurer-healthguide.com

Measuring methods: Temperature measurement in the anus (rectal)

This method of measurement is the most reliable and most accurate. Guide the tip of the thermometer carefully 2 to 3 cm into the anus.

Measurement in the mouth cavity (oral)

For this purpose, guide the tip of the thermometer carefully into one of the heat pouches beneath the tongue, to the left or the right of the root of the tongue.

Measurement of temperature in the armpit (axillary)

This method of measurement is relatively inaccurate. Therefore, it cannot be recommended from a medical point of view.

Mode of measurement	Recommended time required for measurement	Mean divergence to the rectal method
in the anus	until the acoustic signal	–
in the oral cavity	until the acoustic signal	–0,4 °C to –1,5 °C
in the armpit	minimum 5 minutes	–0,7 °C to –2,0 °C

Use

First remove the protective foil from the display. For switching on, briefly press the on/off key. A short acoustic signal will announce that the thermometer is switched on. The thermometer will carry out a self-test for about 2 seconds. During the selftest all segments of the display will appear. Then the reference reading (37 °C) appears. Then the measurement symbol “°C” flashes and “Lo” is shown on the display. The thermometer is now ready for

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